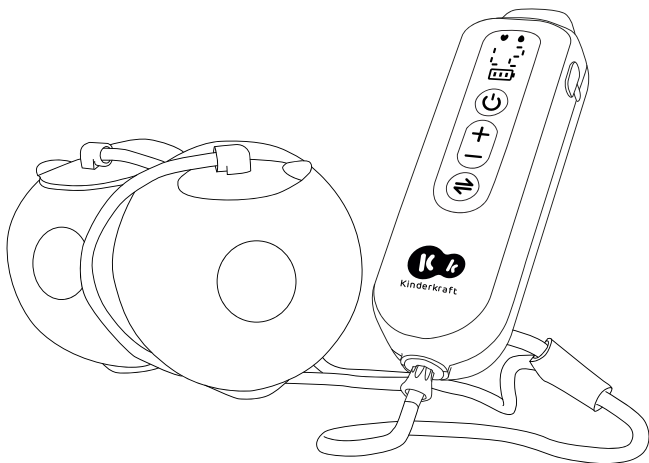
GENTLEFLOW USER GUIDE

(AR) مضخة حليب كهربائية
(CS) ELEKTRICKÁ ODSÁVAČKA MATEŘSKÉHO MLÉKA
(DE) ELEKTRISCHE MILCHPUMPE
(EN) ELECTRIC BREAST PUMP
(ES) SACALECHES ELÉCTRICO
(FR) TIRE-LAIT ÉLECTRIQUE
(HU) ELEKTROMOS MELLSZÍVÓ
(IT) TIRALATTE ELETTRICO
(NL) ELEKTRISCHE BORSTKOLF
(PL) LAKTATOR ELEKTRYCZNY
(PT) BOMBA TIRA-LEITE ELÉTRICA
(RU) ЭЛЕКТРИЧЕСКИЙ МОЛОКООТСОС
(SK) ELEKTRICKÁ ODSÁVAČKA MLIEKA
(SV) ELEKTRISK BRÖSTPUMP

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EN

Dear customer!

Thank you for choosing a product by Kinderkraft.

Our designs are child-oriented – safety and quality always comes first, stay comfortable knowing that you have made the best choice.

PLEASE READ THE USER GUIDE CAREFULLY BEFORE USING THE DEVICE.

WARNINGS

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

Use only an external supply with the following specifications: 5,0 V --- ; 1,0 A, with USB type C plug.

Do not expose a battery pack or appliance to fire or excessive temperature.

Metal objects can cause short circuits if they come into contact with the battery terminals.

This appliance contains batteries that are non-replaceable. When the battery is at end of life, the appliance shall be properly disposed of.

Do not put the breast pump main unit into water or other solutions to clean (or into the sterilizer disinfection).

Can not be used when charging.

The device is intended for household use only and is not suitable for commercial use.

Technical data:

Operating temperature range: 0°C - 40°C, Charging temperature range: 20°C - 30°C

1. Parts List (Fig. A):

No.	Part name	Quantity
1	Lid	2
2	Membrane	8
3	Milk collector	2
4	Double-sealed flange	2
5	Check valve	8

6	Main unit	1
7	Hose connection port	1
8	Silicone tube connector	1
9	Long silicone tube	1
10	Short silicone tube	2
11	L-type connector	1
12	Silicone funnel 24 mm	2
13	Silicone funnel 19 mm	2
14	Silicone funnel 17 mm	2
15	Silicone funnel 24 mm	2
16	Breast pump storage bag	1
17	Lanyard	1

2. Assembly:

Attach the membrane (2) to the double-sealed flange (4) (Fig. B). Install the check valve (5) into the flange (4) according to the direction shown in Fig. C. Then connect to the milk collector (3) (Fig. D) and press firmly with both hands to ensure all parts are securely connected (Fig. E). Next, connect the long silicone tube (9) to the hose connection port (7) and the tube connector (8). Connect the short silicone tubes (10) to the tube connector (8) and the lids (1) (Fig. F).

Double use: Insert the hose connection port (7) into the main unit (6), then place two lids (1) onto the corresponding positions of the two milk collectors (3) (Fig. G).

Single use: Insert the hose connection port (7) into the main unit (6). Place one lid (1) onto the milk collector (3), and insert the other lid into the silicone tube connector (8) together with the L-type connector (11) as shown in Fig. H.

3. Charging the Breast Pump (Fig. K):

Open the charging port on the main unit and insert the USB Type-C charging cable. Then connect the adapter (5V / 1A) to a power outlet. During charging, the power indicator lights up. After disconnecting the power supply, the charging display turns off. Fig. L shows the charging process.

Notes:

- Each operating cycle lasts 30 minutes. The device will automatically shut down after 30 minutes.
- When the battery level is low, the display will flash until the device automatically turns off after the battery is depleted.
- Charging time is approximately 2 hours.

4. Description of the Main Unit (Fig. M):

1	Stimulation mode indicator	5	Power button
2	Expression mode indicator	6	Increase suction level
3	Display	7	Decrease suction level
4	Battery level indicator	8	Mode selection

5. Operating Modes:

	Stimulation mode: setting range Level 1–Level 9
	Mixed mode: setting range Level 1–Level 9

Expression mode: setting range Level 1–Level 9

6. Using the Breast Pump:

- The size of the funnel does not affect the amount of milk collected, but it affects comfort.
- This product is supplied with silicone funnels in four sizes (17, 19, 21, and 24 mm). Make sure to use the most comfortable fit.
- The funnel should be aligned with the center of the nipple (Fig. N). Ensure the funnel is correctly positioned and pressed firmly against the breast so that there are no gaps (Fig. I). Repeat on the other side.
- Place the device inside a nursing bra. The main unit can be attached to the lanyard and worn around the neck. Start pumping milk (Fig. J). The device can be used while sitting or standing.
- If you feel pain or discomfort, turn off the device and adjust the funnel position.

7. Suction Level Adjustment:

During operation, the suction level can be adjusted at any time.

- Briefly press the “+” or “-” button to increase or decrease the level by one.
- Press and hold the “+” or “-” button to speed up adjustment.
- Changing the suction level will briefly pause the operation.
- Adjusting the suction level during the operating cycle does not affect the total cycle time of 30 minutes.
- The product has a mode memory function; the default mode will switch to the last used mode.

8. Pouring Milk:

After pumping, remove the breast pump and point the funnel upward to avoid spilling milk. Prepare a bottle for milk storage and pour the milk according to the correct pouring method shown in Fig. O.

9. Storage of Breast Milk:

Temperature – range	Storage time for a healthy baby	Storage time for a sick or premature baby
Temperature up to 15°C	Up to 24 hours	Up to 1 hour
Cooling bag with ice packs – for transport only	Up to 24 hours	Up to 24 hours
	2–5 days	96 hours
Refrigerator (back wall) (+4°C) – do not store in the refrigerator door	2 weeks	1 week
Refrigerator freezer compartment (shared door of refrigerator and freezer) (–10°C)	3–4 months	Several weeks
Fridge-freezer (separate doors for refrigerator and freezer) (–14°C)	6 months or longer	3 months

- Because breast milk expands when frozen, it is recommended to fill bottles only 3/4 full before freezing.
- It is recommended that each stored portion be approximately 180 ml – this allows faster thawing and prevents waste.
- Label each storage container with the date.
- Thaw breast milk for several hours in the refrigerator or in warm water at about 40°C. When thawing in warm water, gently swirl the milk to prevent fat separation.
- Thawed breast milk must not be refrozen and must be consumed within 24 hours.

Note:

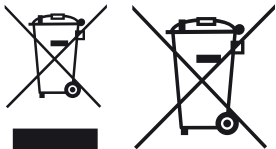
- Do not store breast milk in the refrigerator door.
- To preserve nutrients, frozen milk should be thawed in the refrigerator the night before use.
- Do not heat frozen milk in a microwave, as this may cause nutrient loss and risk burning the baby.
- Do not mix fresh breast milk with thawed breast milk.

10. Cleaning and Disinfection:

Keep the product clean. Do not wash or disinfect the charging cable and the main unit of the breast pump (Fig. P) in water; wipe the surface with a paper towel. In the event of accidental immersion of the device in water or contact with another liquid, stop using it immediately and contact customer service.

All other parts of the breast pump (Fig. P, R) should be washed by hand only using a mild detergent: before first use and after each subsequent use, rinse them under running water with a gentle cleaning agent, then rinse thoroughly with warm water. These parts may be disinfected with steam or in boiling water; after the water reaches boiling point, turn off the heat source and wait three minutes before starting disinfection. After washing, allow the parts to dry completely.

DISPOSAL OF WASTE EQUIPMENT AND BATTERIES

The symbol of the crossed-out wheeled bin placed on equipment, batteries, packaging or accompanying documents indicates that the product must not be disposed of with other waste. It is the user's responsibility to hand over waste equipment and batteries to a designated collection point for proper treatment. Information about the available waste electrical equipment and battery collection system can be found in the shop's information point and in the municipal/commune office.

Each household is a user of electrical and electronic equipment and batteries and therefore a potential producer of waste hazardous to humans and the environment due to the presence of hazardous substances, mixtures and components in the equipment and batteries. On the other hand, waste equipment is a valuable material from which we can recover raw materials such as copper, tin, glass, iron and others. Proper handling of waste equipment and batteries prevents negative consequences for the environment and human health!

11. WARRANTY

The full text of the Terms and Conditions of the Warranty is available on the website WWW.KINDERKRAFT.COM

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