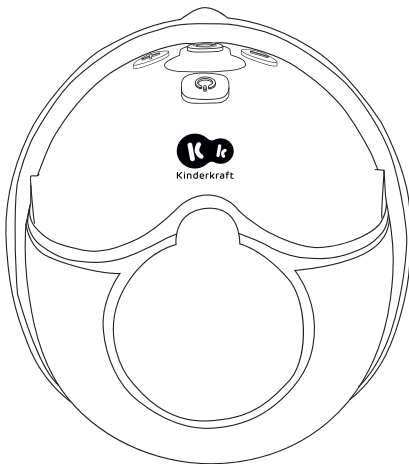
MAMA FLOW USER GUIDE

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EN

Dear customer!

Thank you for choosing a product by Kinderkraft.

Our designs are child-oriented – safety and quality always comes first, stay comfortable knowing that you have made the best choice.

PLEASE READ THE USER GUIDE CAREFULLY BEFORE USING THE DEVICE.

WARNINGS

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

Use only an external supply with the following specifications: 5,0 V --- ; 1,0 A, with USB type C plug.

Do not expose a battery pack or appliance to fire or excessive temperature.

Metal objects can cause short circuits if they come into contact with the battery terminals.

This appliance contains batteries that are non-replaceable. When the battery is at end of life, the appliance shall be properly disposed of.

Do not put the breast pump main unit into water or other solutions to clean (or into the sterilizer disinfection).

Can not be used when charging.

The device is intended for household use only and is not suitable for commercial use.

Technical data:

Operating temperature range: 0°C - 40°C, Charging temperature range: 20°C - 30°C

1. List of Parts (Fig. A):

Lp.	Part Name	Quantity
1	Main unit	1
2	Milk collector	1
3	Flange	1
4	Cover	1
5	Membrane cover	1
6	Membrane	4
7	Check valve 4	4
8	Silicone insert 19 mm	1
9	Silicone insert 21 mm	1
10	Silicone insert 17 mm	1
11	Storage bag for breast pump	1

2. Assembly:

Insert the check valve (7) into the flange (3), paying attention to the direction. Ensure that the valve is fully inserted and properly oriented (Fig. B).
 Insert the flange (3) into the milk collector (2). Make sure the notch at the top of the flange is facing the milk outlet. Check that the flange (3) is fully seated around the entire milk collector (2); otherwise, milk may leak (Fig. C).
 Insert the membrane (6) into the membrane cover (5) (Fig. D).
 Slide the membrane cover (5) into the milk collector (2) in the indicated direction and press it in fully. The tube of the membrane cover must fit into the groove of the milk collector (Fig. E).
 Then install the main unit (1) onto the milk collector (2) in the indicated direction (Fig. F). Ensure the main unit is properly mounted.

3. Charging the Breast Pump:

Use two fingers to remove the main unit (1) from both sides and detach it from the milk collector (2) (Fig. G).
 Connect the USB Type-C charging cable to the charging port in the top right corner of the main unit, then connect the power adapter (5 V, 1 A) to the socket to begin charging (Fig. H).
 The display remains on during charging. After charging is complete, the device enters sleep mode. After disconnecting the power adapter, the device switches off.

Battery Indicator Description

Press the power button once when the device is switched off to check the battery level.

	Fully charged		The icon indicates a battery level of 40–10%.
	The icon indicates a battery level of 100–70%.		When the value drops below 10%, the icon will flash until the battery reaches 0.
	The icon indicates a battery level of 70–40%.		

- One full charge allows approximately 5–6 cycles. One working cycle lasts about 30 minutes.
- When the battery level is low, the display will flash until the device automatically switches off.
- Charging time is approximately 2 hours.
 Note: The device cannot be used while charging.

4. Control Panel Description (Fig. I):

1	Power button	5	Expression mode
2	Massage mode	6	Decrease suction strength
3	Automatic mode	7	Increase suction strength
4	Stimulation mode	8	Mode selection / mode change

Operating the main unit/control panel:

- Press and hold the power button for 2 seconds to switch on the breast pump.
- During the first use, the device starts in automatic mode by default.
- A short press of the power button pauses the current operation, and the display will start flashing.
- Another short press resumes operation, and the display remains lit.
- After 20 seconds of inactivity, the display switches off and the device enters sleep mode.
- Press any button to reactivate the display.
- Press and hold the power button for 2 seconds to switch off the device.
- If the device remains paused for 5 minutes without any action, it will switch off automatically.

5. Breast Pump Modes:

Press the mode selection button to switch between functions.

	Automatic mode: 5 short + 10 long suction (suction strength level 5–9)
	Massage mode: short suction (suction strength level 1–9)
	Stimulation mode: 5 short + 1 long suction (suction strength level 1–9)
	Expression mode: long suction only (suction strength level 1–9)

6. Usage Instructions:

- The size of the insert does not affect the amount of milk collected but does affect comfort.
- The product includes silicone inserts in 3 sizes (17, 19, and 21 mm). Ensure you use the most comfortable fit.
- The insert should align with the center of the nipple (Fig. J). Make sure the selected insert fits correctly.
- Place the device inside a nursing bra. It can be used both while sitting and standing.
- If you experience pain or discomfort, switch off the device and adjust the position of the insert (Fig. J). Check that no gap exists between the breast and the silicone insert. Restart the device on the first suction level. If pain persists, stop using the product immediately and consult a doctor.

7. Suction Level Adjustment:

- During operation, you can change the suction level at any time. A short press of the “+” / “-” button will increase or decrease the level by one step. After pressing “+” or “-”, the main unit will temporarily pause operation.
- Pressing and holding the “+” or “-” button will speed up the change of the suction level.
 - Changing the suction level will cause a brief interruption of device operation.
 - Adjusting the suction level during the working cycle will not affect its total duration of 30 minutes.
 - The product features a mode memory function – the default mode will switch to the last used mode.

8. Pouring the Milk:

Hold the main unit (1) with two fingers on both sides and lift it from the milk collector (2) (Fig. G). Then position the milk collector so that the flange is at the top. Tilt it by approx. 45°. Place the milk outlet as close as possible to the bottle neck and pour the milk out (Fig. K).

9. Storage of Breast Milk:

Temperature – range	Storage time for a healthy baby	Storage time for a sick or premature baby
Temperature up to 15°C	Up to 24 hours	Up to 1 hour
Cooling bag with ice packs – for transport only	Up to 24 hours	Up to 24 hours
	2–5 days	96 hours
Refrigerator (back wall) (+4°C) – do not store in the refrigerator door	2 weeks	1 week
Refrigerator freezer compartment (shared door of refrigerator and freezer) (–10°C)	3–4 months	Several weeks
Fridge-freezer (separate doors for refrigerator and freezer) (–14°C)	6 months or longer	3 months

1. Because breast milk expands when frozen, it is recommended to fill bottles only 3/4 full before freezing.
2. It is recommended that each stored portion be approximately 180 ml – this allows faster thawing and prevents

- waste.
3. Label each storage container with the date.
 4. Thaw breast milk for several hours in the refrigerator or in warm water at about 40°C. When thawing in warm water, gently swirl the milk to prevent fat separation.
 5. Thawed breast milk must not be refrozen and must be consumed within 24 hours.

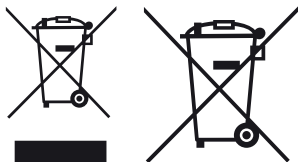
Note:

- Do not store breast milk in the refrigerator door.
- To preserve nutrients, frozen milk should be thawed in the refrigerator the night before use.
- Do not heat frozen milk in a microwave, as this may cause nutrient loss and risk burning the baby.
- Do not mix fresh breast milk with thawed breast milk.

10. Cleaning and Disinfection:

Keep the product clean. Do not wash or disinfect the charging cable or the main unit in water; wipe the surface with a paper towel. If the device is accidentally submerged in water or comes into contact with another liquid, stop using it immediately and contact customer service. Wash all other breast pump components only by hand using a mild detergent: Before first use and after every use, rinse under running water, wash with mild soap, and rinse thoroughly with warm water. These components may be disinfected with steam or in boiling water; after the water reaches boiling, turn off the heat and wait three minutes before disinfection. Allow all parts to dry completely before reassembly. It is recommended to replace the membrane and the check valve every 3 months.

DISPOSAL OF WASTE EQUIPMENT AND BATTERIES

The symbol of the crossed-out wheeled bin placed on equipment, batteries, packaging or accompanying documents indicates that the product must not be disposed of with other waste. It is the user's responsibility to hand over waste equipment and batteries to a designated collection point for proper treatment. Information about the available waste electrical equipment and battery collection system can be found in the shop's information point and in the municipal/commune office. Each household is a user of electrical and electronic equipment and batteries and therefore a potential producer of waste hazardous to humans and the environment due to the presence of

hazardous substances, mixtures and components in the equipment and batteries. On the other hand, waste equipment is a valuable material from which we can recover raw materials such as copper, tin, glass, iron and others. Proper handling of waste equipment and batteries prevents negative consequences for the environment and human health!

11. WARRANTY

The full text of the Terms and Conditions of the Warranty is available on the website WWW.KINDERKRAFT.COM

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